






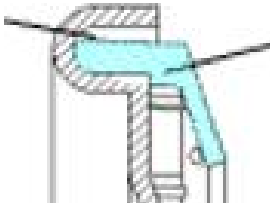




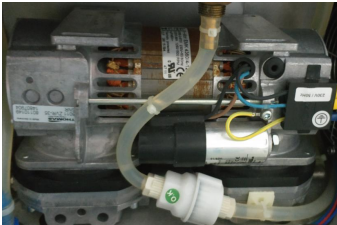





	Tuttnauer		SHINVA	
Brand				
Model	EZ9Plus	EZ11Plus	T24-FA	T45-FA
Chamber Dimensions Ø x L in (mm)	9"×19.8" (230×504)	11"×19.8" (280×504)	9.8"×17.7" (250×450)	12.4"×24.3" (316×618)
Volume/L	19.8	28.5	24	45
Power (W)	1400W		2600W	5800W
Sterilizer Type	B		B	
Overall Dimensions DxWxH in (mm)	24.8"×19.1"×15.1" (630×485×384)	24.8"×20.9"×17.3" (630×530×440)	26"×20.9"×17.7" (660×530×450)	33.7"×25.2"×21.7" (855×640×550)
Chamber material	316L		316L	
Chamber Outer surface heating			Heating film	
Insulation materials	Fibreglas		Viscose fiber insulation layer	
Door locking method				
	Manual locking		Automatic locking	
Safety interlocking			 	
	Electromagnetic lock		Electromagnetic lock+pressure lock	
Door panel material	304		316L	

Gasket		
	Self expanding type	Self expanding type
Display		
	Multi-color display	Black and white display
Button		
	Mask-style	Inductive
Ultimate vacuum	$\leq -90\text{kpa}$	$\leq -90\text{kpa}$
Vacuum system		
	Vacuum pumps	Circulation pump + ejector (patent technology)
Printer	Optional (thermal printer)	Standard (thermal printer)
Water source	Inner cycle	Inner cycle
Number of tanks		
	2 (water purification tank + sewage tank)	1 (circulating water tank)
Water quality monitoring	Yes	Yes
USB	Standard	Standard

Optional Configurations	Paper-plastic bag holder	Paper-plastic bag holder
	Tray picker (metal)	Tray picker (metal)
	Tray	Tray
	Package loading rack 2 F/4 F	Package loading rack 2 F/4 F
		Water dispenser
Program	<p>Program name : Unwrapped Instruments (FDA)</p> <p>Sterilization Temp.: 270°F/132°C</p> <p>Sterilization Time: 3min</p> <p>Dry Time: 1min</p> <p>Total cycle Time: EZ9PLUS: 24min; EZ11PLUS: 28min</p>	<p>Program name: B134Univ. (FDA)</p> <p>Sterilization Temp.: 273°F/134°C</p> <p>Sterilization Time: 4min</p> <p>Dry Time: 20min</p> <p>Total cycle Time: T24-FA: 55min; T45-FA: 58min</p>
	<p>Program name : Wrapped Instruments, Pouches (FDA)</p> <p>Sterilization Temp.: 270°F/132°C</p> <p>Sterilization Time: 4min</p> <p>Dry Time: 30min</p> <p>Total cycle Time: EZ9PLUS: 51min; EZ11PLUS: 55min</p>	<p>Program name: B121Univ. (FDA)</p> <p>Sterilization Temp.: 250°F/121°C</p> <p>Sterilization Time: 20min</p> <p>Dry Time: 20min</p> <p>Total cycle Time: T24-FA: 65min; T45-FA: 68min</p>
	<p>Program name : Unwrapped Delicate Instruments (FDA)</p> <p>Sterilization Temp.: 250°F/121°C</p> <p>Sterilization Time: 30min</p> <p>Dry Time: 1min</p> <p>Total cycle Time: EZ9PLUS: 47min; EZ11PLUS: 50min</p>	<p>Program name: B-Quick (FDA)</p> <p>Sterilization Temp.: 273°F/134°C</p> <p>Sterilization Time: 4min</p> <p>Dry Time: 1min</p> <p>Total cycle Time: T24-FA: 26min; T45-FA: 28min</p>
	<p>Program name: Handpieces (FDA)</p> <p>Sterilization Temp.: 270°F/132°C</p> <p>Sterilization Time: 4min</p> <p>Dry Time: 30min</p> <p>Total cycle Time: EZ9PLUS: 55min; EZ11PLUS: 60min</p>	<p>Program name: N-Quick (FDA)</p> <p>Sterilization Temp.: 273°F/134°C</p> <p>Sterilization Time: 4min</p> <p>Dry Time: 1min</p> <p>Total cycle Time: T24-FA: 16min; T45-FA: 18min</p>
	<p>Program name: Chamber Brite Cleaning</p> <p>Sterilization Temp.: 270°F/132°C</p> <p>Sterilization Time: 3min</p>	<p>Program name: User</p> <p>Sterilization Temp.: 273°F/134°C</p> <p>Sterilization Time: 4min</p> <p>Dry Time: 20min</p> <p>Total cycle Time: T24-FA: 55min; T45-FA: 58min</p>
	<p>Program name: Custom A</p> <p>Sterilization Temp.: 270°F/132°C</p> <p>Sterilization Time: 4min</p> <p>Dry Time: 30min</p> <p>Total cycle Time: EZ9PLUS: 54min; EZ11PLUS: 56min</p>	<p>Program name: Preheat</p> <p>Sterilization Temp.: 273°F/134°C</p> <p>Sterilization Time: 4min</p> <p>Dry Time: 0.5min</p> <p>Total cycle Time: T24-FA: 15min; T45-FA: 17min</p>
	<p>Program name: Custom B</p> <p>Sterilization Temp.: 250°F/121°C</p>	<p>Program name: Drying</p> <p>Dry Time: 20min</p>

	<p>Sterilization Time: 30min Dry Time: 1min Total cycle Time: EZ9PLUS: 44min; EZ11PLUS: 47min</p>	<p>Total cycle Time: T24-FA: 20min; T45-FA: 20min</p>
		<p>Program name: Cleaning Washing time: 10min Total cycle Time: T24-FA: 10min; T45-FA: 10min</p>