

How to set up TELSCOPE TELEHEALTH SYSTEM

1. **Download** the TelScope app
2. **Apply** the metal clip to your smart device
3. **Attach** the TelScope handle to your smart device



ANDROID



iOS



1. DOWNLOAD

Download the TelScope app to your smart device.

1. **Download** the TelScope app with the QR codes above or by visiting hollandhealthcareinc.com/app

2. APPLY

Apply the metal clip to your smart device.

1. **Clean** the device or case surface with the **provided alcohol wipe**
2. **Allow to dry** and wipe off any further dust with a lint free cloth
3. Carefully **remove the red liner** from the clip
4. Apply the metal phone clip **1" directly below the camera in a vertical position**
5. Apply **gentle pressure** for 30 seconds
6. Leave the clip for **24 hours** before it is used with TelScope

3. ATTACH

Attach the TelScope handle to your smart device.

1. Insert the single use **Throat Scope blade** into the TelScope handle
3. Gently slide the TelScope handle **upwards** onto the metal clip to attach it to your smart device
4. Open the TelScope app
5. TelScope is now ready for **oral examination** and **capturing telehealth images**
6. Gently slide the TelScope handle **downwards** to detach