















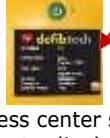









AED Quick Check Guide

Physio-Control (Medtronic)	Visual Inspection	Pad Expiration Date	Notes
CR Plus 	 = OK/Ready  = Replace PadPak  = Service Required		Internal battery lasts approximately 8 years. Older units should be replaced.
Express 	 = OK/Ready  = Replace PadPak  = Service Required		Internal battery lasts approximately 8 years. Older units should be replaced.
LP 500 	 = OK/Ready  = Low Battery  = Service Required		Discontinued – No longer supported. Pads available but batteries are not. Should consider replacement.
Philips Healthcare			
Onsite 	 Flashing Green = OK  Flashing Red = failed self-test (Low battery or bad pads)  No light = battery dead		
FRx 	 Flashing Green = OK  Flashing Red = failed self-test (Low battery or bad pads)  No light = battery dead		
FR2 	 Hour Glass = OK  Red "X" = failed self-test (Low, dead or no battery)		Discontinued – No longer supported. Pads and batteries discontinued 12/2018. Should consider replacement
FR3 	 Flashing Green = OK  Flashing Red = failed self-test (Low battery or bad pads)  No light = battery dead		
Forerunner 	 Hour Glass = OK  Red "X" = failed self-test (Low, dead or no battery)		Discontinued – No longer supported. Pads available but batteries are not. Should consider replacement.

Heartsine (Physio – Control)			
350P	 Flashing Green = OK Flashing Red = failed self-test or Low battery No light = battery dead		
360P	 Flashing Green = OK Flashing Red = failed self-test or Low battery No light = battery dead		
450P	 Flashing Green = OK Flashing Red = failed self-test or Low battery No light = battery dead		
Cardiac Science			
Firstsave	Discontinued – Replace	Discontinued – Consumables no longer available	
9200D	Discontinued - Replace	Discontinued – Consumables no longer available	
G3	 Solid Green = OK Solid red = failed self-test or Low battery	 	
G3 Plus	 Solid Green = OK Solid red = failed self-test or Low battery	 	
G5	 Solid Green = OK Solid red = failed self-test or Low battery * Battery power level can be viewed on LCD screen	 	
Defibtech (Cintas Reviver)			
View	Flashing Green = OK Flashing Red = failed self-test or Low battery No light = battery dead	  Press center soft key to display	
Lifeline	 Flashing Green = OK Flashing Red = failed self-test, low 9V or lithium battery No light = battery dead		
Welch Allyn			
AED 10	Discontinued - Replace	Discontinued – Accessories no longer available	Support, pads and batteries no longer available
ZOLL			
AED Plus	   Green Check = Pass Red "X"/ "beeping" = Fail	  Green tab extending from slot in cover	